

Case Study 8: Infected Trauma Wound

A 67-year-old male presented with an infected (moderate growth of *Enterococcus faecalis*) trauma wound. After adequate debridement, V.A.C. VeraFlo™ Therapy was initiated using V.A.C. VeraFlo™ Dressing. Normal saline was initially used; 10mL was instilled, followed by a soak time of 15 minutes. Instillation was repeated every 3.5 hours, followed by continuous negative pressure at -125 mmHg for 7 days. The instillate was changed to Lactated Ringer's Solution at first dressing change. No complications occurred during therapy, and the wound was clean and closed by primary intention.



A. Wound at initial presentation



B. First dressing change followed by surgical debridement



C. Second dressing change followed by surgical debridement



D. Wound after 7 days of V.A.C. VeraFlo™ Therapy